

Name : Fabric 340 Sterilisation Wrap

Product Code: REL340Gxxxx, REL340Bxxxx

Name of Manufacturer: Clinipak Ltd

Address: Beech House

Knaves Beech Business Centre

Loudwater

High Wycombe

Bucks

HP10 9SD

Tel: +44 (0)1628 810626

Fax: +44 (0)1628 810760

Website: clinipak.co.uk

Intended Use

The intended use of Fabric 340 Sterilisation Wrap is as a microbial barrier sterilisation wrap for use in sterile barrier packaging systems which are designed for the sterilisation of surgical instruments by steam, ethylene oxide or gamma radiation methods. It is used in combination with a second sterilisation wrap. The second sterilisation wrap can be another sheet of Fabric 340 sterilisation wrap or a different type of sterilisation wrap such as a nonwoven wrap or a paper wrap.

Fabric 340 Sterilisation Wrap is intended to be used by professional healthcare personnel working in sterile services (or similar) departments.

Single Use

Fabric 340 Sterilisation Paper is single use only.

Storage Conditions

It is recommended that Fabric 340 Sterilisation Wrap is stored under the conditions stated on the label of the box. If this is not possible, it should be prevented from getting wet or from being exposed to high temperatures (>40degC).

Precautions

Fabric 340 Sterilisation Wrap should be stored away from any sources of ignition..

Disposal

Unused Fabric 340 Sterilisation wrap may be disposed of in the same way that the user normally disposes of wrap. It is suitable for recycling in the waste wrap stream.


Used Fabric 340 Sterilisation Wrap should be disposed of by taking into account possible microbial contamination. The users should reference their own procedures in this matter.



Wrapping with Fabric 340 Sterilisation Wrap

- 1) Blue and Green Fabric 340 Sterilisation Wrap can be used in the same way as any other standard Tray Wrap.
- 2) Fabric 340 sterilization wrap should always be used in combination with at least one other tray wrap. When used with a non-woven or SMS-type Tray Wrap, the Fabric 340 Sterilisation Wrap can be used as either the inner or the outer wrap.
- 3) Any of the tray wrapping techniques can be used in either a sequential or simultaneous wrapping procedure.



Below are examples of 2 well-known wrapping techniques - **Parcel Fold and Envelope fold**. The examples use Green Fabric 340 Sterilisation Wrap as the inner wrap and the Blue Fabric 340 Sterilisation Wrap as the outer wrap:

Parcel Fold

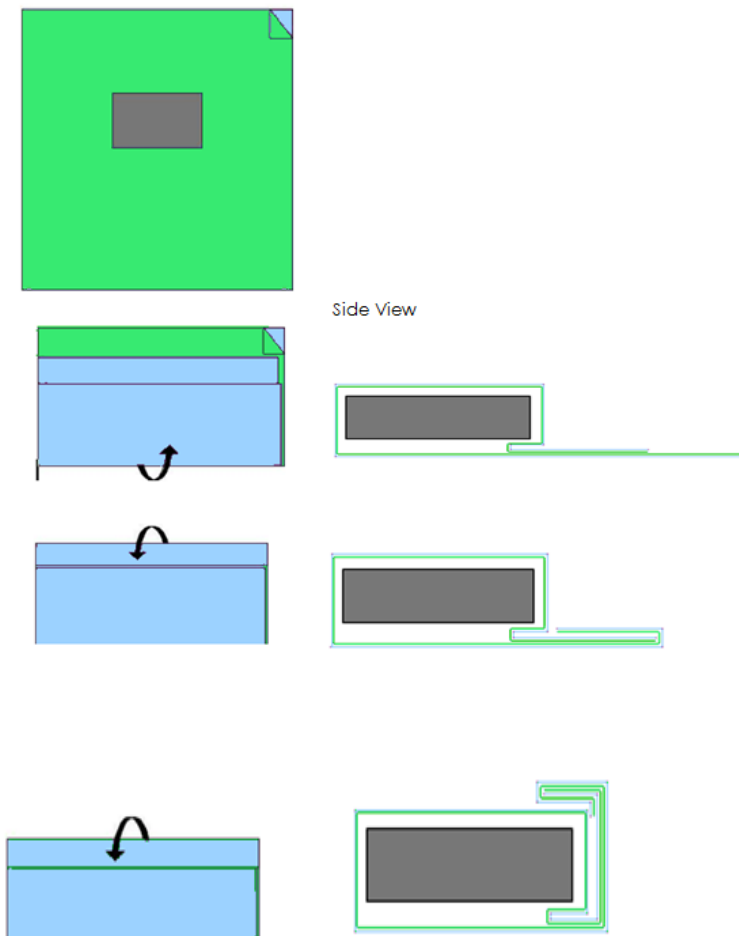

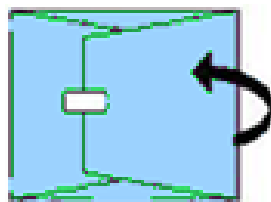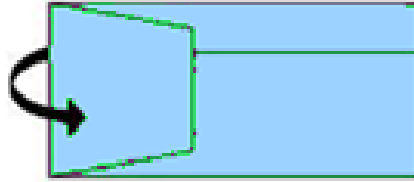

Envelope Fold

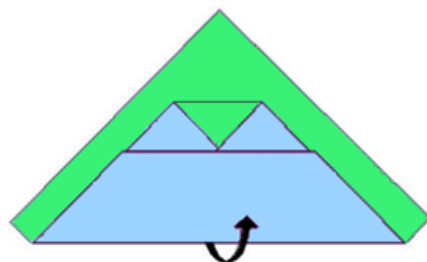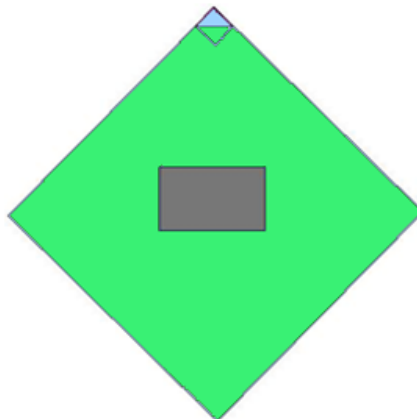

Side View



