

Breakaway Tags

INTENDED USE:

Key Surgical® Breakaway Tags are used to identify instruments, devices, and other equipment.

PRECAUTION:

The LR-322, LR-323, LR-324, LR-325, LR-327, and LR-329 cannot be sterilized.

INSTRUCTIONS FOR USE:

1. Breakaway Tags do not need to be cleaned or disinfected before use.
2. Affix tag to instrument, device or other equipment by cinching closed until tight.
3. Tags can be sterilized according to the parameters detailed below.
4. Tags are single use and can be sterilized only once.
5. Tags can be broken and removed by hand or with the use of cutters.

STERILIZATION:

The Breakaway Tags are provided non-sterile. They have been validated for steam sterilization at the following cycle parameters:

<i>Cycle Type</i>	<i>Temperature</i>	<i>Minimum Time – Full Cycle</i>
Pre-Vacuum	132° C (270° F)	4 minutes

They have been validated for the following STERRAD® Systems and cycles:

<i>STERRAD® System and Cycle within the United States</i>	<i>STERRAD® System and Cycle outside United States</i>
STERRAD® 50	STERRAD® 50
STERRAD® 200	STERRAD® 200 Short cycle
STERRAD® 100S	STERRAD® 100S Short cycle
STERRAD® NX Advanced cycle	STERRAD® NX Advanced cycle
STERRAD® NX Standard cycle	STERRAD® NX Standard cycle
STERRAD® 100NX Standard cycle	STERRAD® 100NX Standard cycle

DISPOSAL:

There are no special disposal instructions.



MANUFACTURER: KEY SURGICAL LLC
8101 WALLACE ROAD / EDEN PRAIRIE, MN 55344 / USA
PH: 800.541.7995 KEYSURGICAL.COM



MDSS GMBH
SCHIFFGRABEN 41, D-30175 / HANNOVER, GERMANY

